


# FUNGITELL® ASSAY


## TEST PROCEDURE OUTLINE

### SET-UP

**1** Reconstitute glucan stock and make standards.




**2** Pipette 5  $\mu$ L of sample into Pyroplate wells.

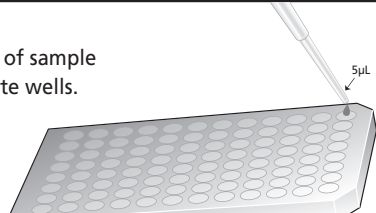


### TEST SAMPLES


**3** Add 20  $\mu$ L of alkaline pretreatment solution into Pyroplate sample wells.

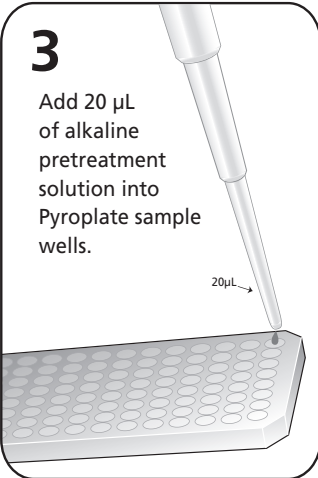


**4** Incubate at 37° C for 10 minutes.

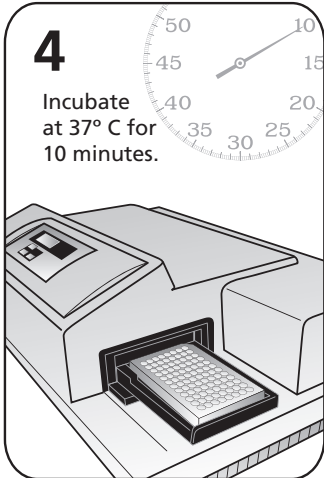


**5** Reconstitute Fungitell® Reagent.




**6** Add 25  $\mu$ L standards to Pyroplate.

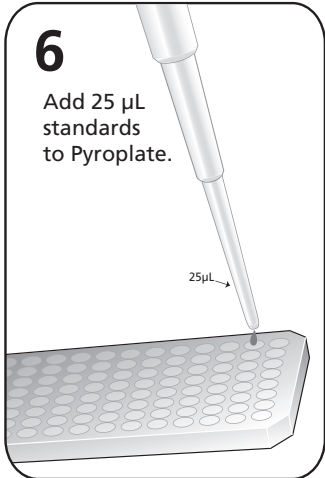


**7** Add 100  $\mu$ L of Fungitell® lysate reagent into Pyroplate wells.

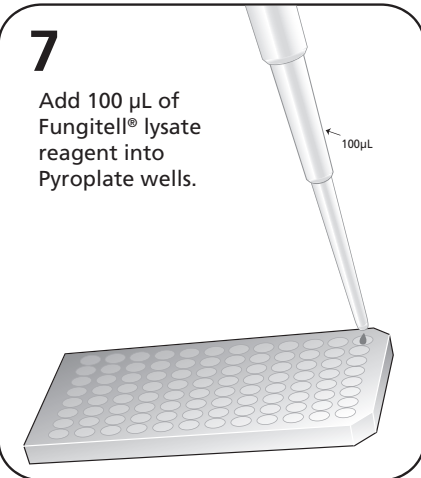

**8** Incubate and collect kinetic data at 37° C for 40 minutes.

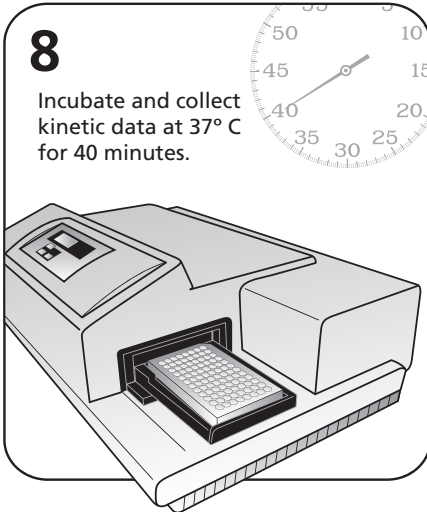

**9** Read report data.

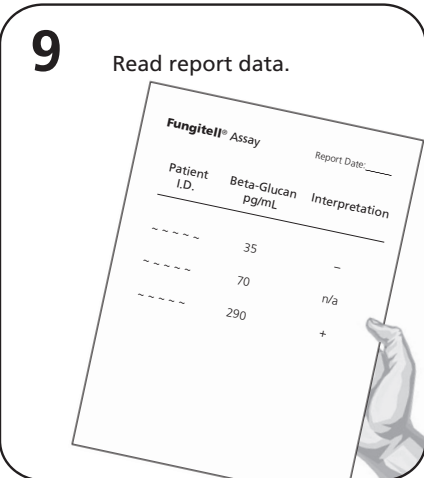

**NOTE:** For complete test procedure refer to Fungitell® Instructions For Use (IFU).



ASSOCIATES OF  
**CAPE COD**  
INCORPORATED

*Specialists in Endotoxin and Glucan Detection*

**Corporate Headquarters**

**Associates of Cape Cod, Inc.**

124 Bernard E. Saint Jean Drive  
East Falmouth, MA 02536-4445 USA

Tel: (888) 395-ACC1(2221) or

(508) 540-3444

Fax: (508) 540-8680

[www.acciusa.com](http://www.acciusa.com)

Customer Service:

[custservice@acciusa.com](mailto:custservice@acciusa.com)

Technical Service:

[techservice@acciusa.com](mailto:techservice@acciusa.com)

Contract Test Service:

[testservice@acciusa.com](mailto:testservice@acciusa.com)

**United Kingdom**

**Associates of Cape Cod Int'l., Inc.**

Deacon Park, Moorgate Road

Knowsley, Liverpool L33 7RX

United Kingdom

Tel: (44) 151-547-7444

Fax: (44) 151-547-7400

E-mail: [info@acciuk.co.uk](mailto:info@acciuk.co.uk)

[www.acciuk.co.uk](http://www.acciuk.co.uk)

Company Registration Number:

BR002906

**Germany**

**Associates of Cape Cod Europe GmbH**

Opelstrasse 14

D-64546 Mörfelden-Walldorf

Germany

Tel: (49) 61 05-96 10 0

Fax: (49) 61 05-96 10 15

E-mail: [service@acciusa.de](mailto:service@acciusa.de)

[www.acciusa.de](http://www.acciusa.de)